




RibLoc[®] U+

Chest Wall Plating System

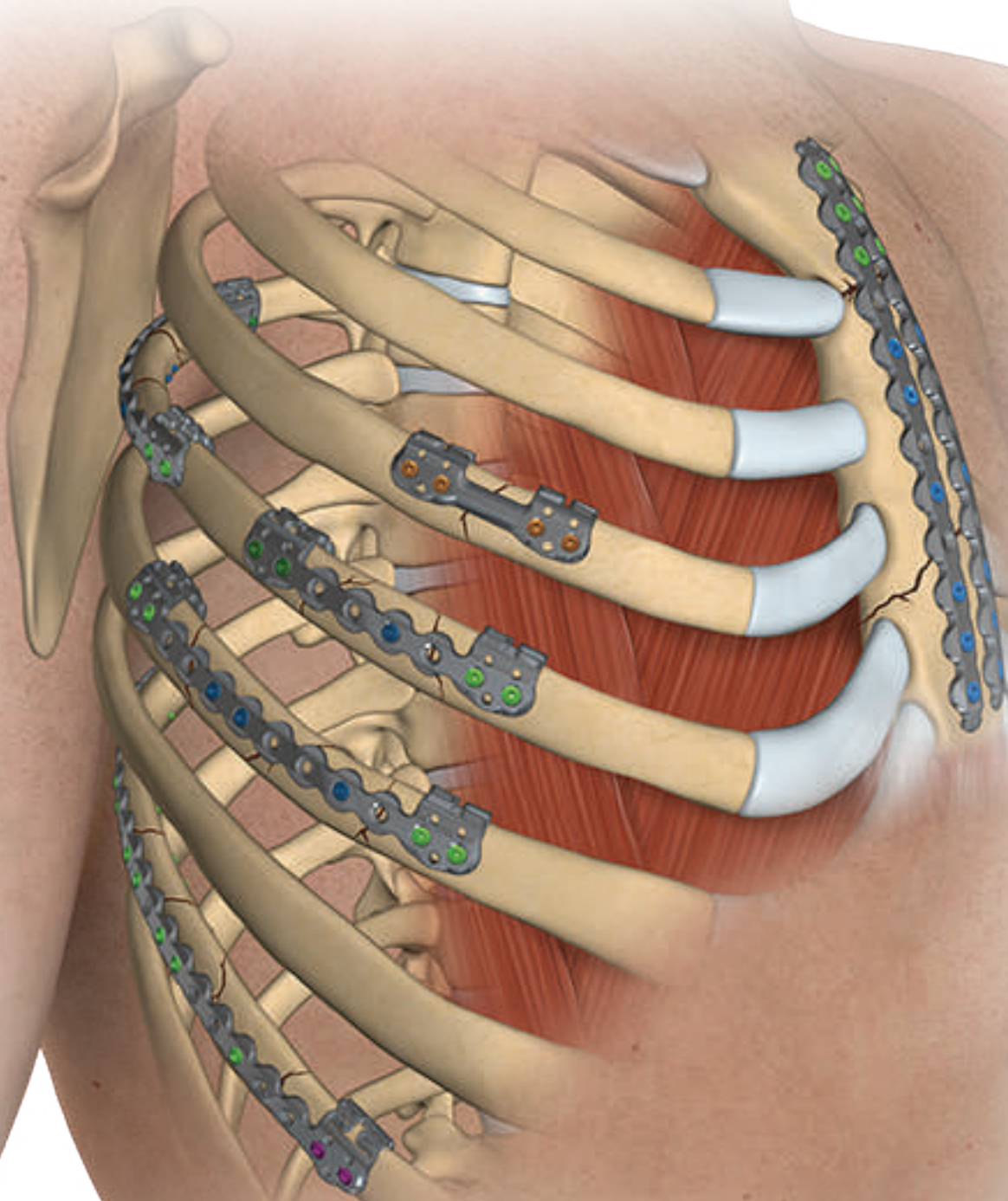


Innovative Solutions for Challenging Thoracic Procedures

RibLoc® U+

Chest Wall Plating System

ACUTE Innovations® has developed U-shaped plate technology to provide stability with anterior and posterior locking screws while avoiding the neurovascular bundle. The RibLoc system does not rely on the rib cortex alone for fixation; instead, the screw locks into the anterior and posterior aspects of the plate and rib to provide added stability¹. Using this technology, the U Plus system provides a solution for rib fracture repair that is stable, efficient, versatile, and customizable to each patient and situation. The low-profile instrumentation of U Plus 90 facilitates fixation of subscapular, posterior, and anterolateral fractures through minimally invasive approaches.

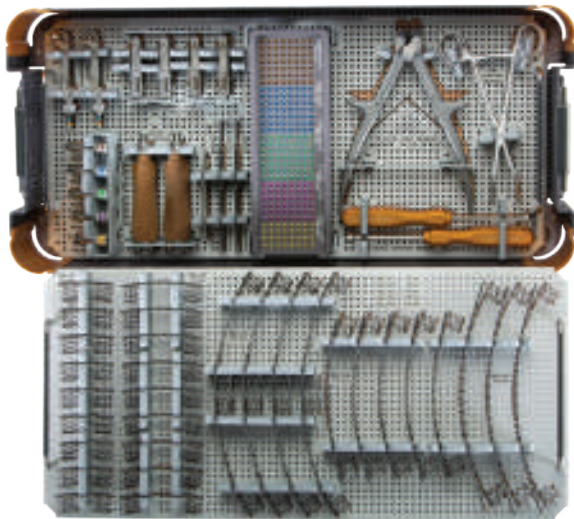

U PLUS CHEST WALL PLATING SYSTEM

The RibLoc U Plus Chest Wall Plating System offers straight-forward, color-coded instrumentation for ease of use. It also includes:

- | U-clips that compress to match the rib thickness
- | Plates that address diverse fracture patterns
- | Custom bending instrumentation
- | Multi-purpose instrumentation
- | NEW implant for sternal fracture repair

U PLUS 90 INSTRUMENTATION

The RibLoc U Plus 90 Instrumentation works in conjunction with the U Plus system providing power for limited access areas to ensure efficiency in the O.R.

Features include:

- | Subscapular access as high as the second rib²
- | Power for installation, including screw insertion
- | LED light that improves visibility
- | Torque limits






14mm

6mm

CUSTOM FIT TO THE RIB

The RibLoc U Plus plate was designed to be customizable to the individual patient. The plates range in length from 50mm-215mm, which allows the surgeon to address a broad range of fracture patterns. The advanced design of the plate offers a compressible U-clip for a perfect fit to a broad range of rib thicknesses (6mm-14mm).

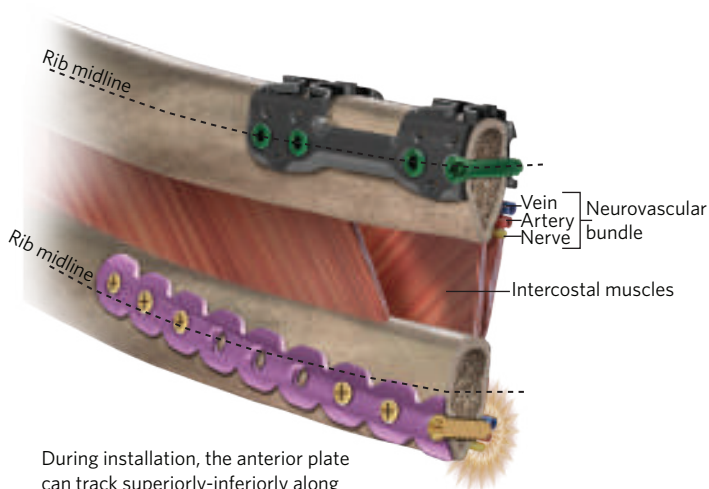

U PLUS PLACEMENT ON THE RIB


The U-shape design of the U Plus plate aids in consistent and straight-forward placement through the thickest portion of the rib while avoiding the neurovascular bundle.³

During installation, the anterior plate can track superiorly-inferiorly along the rib surface, which can lead to neurovascular bundle impingement and reduce screw purchase.

PERFORMANCE OF RIBLOC VERSUS ANTERIOR PLATING

A biomechanical study was performed using cadaver ribs plated with a 4.6cm long RibLoc plate and 4 screws. It was superior in durability to an anterior plate of over twice the length and 6 screws after only 50,000 breathing cycles (typically two days of breathing).¹

This was evaluated based on stiffness loss, as shown in the graph. Additionally, the reduced length of the RibLoc plate may facilitate a less invasive technique.¹

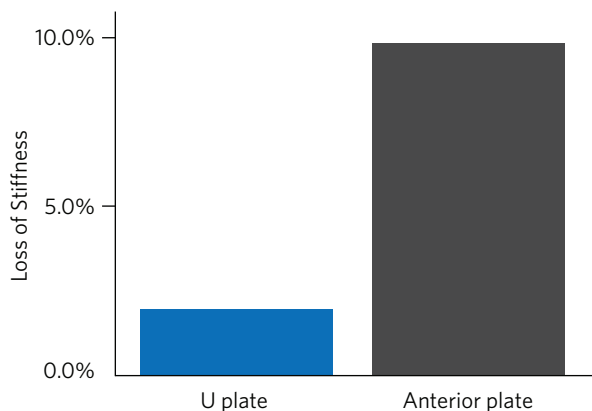

ANTERIOR AND POSTERIOR LOCKING

The RibLoc® system does not rely on the rib cortex alone for fixation; instead, the screw locks into the anterior and posterior aspects of the plate and rib to provide added stability.¹

The U-clips are designed to minimize the stress on the rib by distributing physiologic loads over a greater surface area. While the screws engage and fixate to the cortices of the bone, they also lock both anteriorly and posteriorly into the plate. An additional benefit to the U-clip design is that fewer screws are needed to anchor the plate to the bone, lessening the degree of dissection needed on the rib itself⁴, as well as reducing cost.

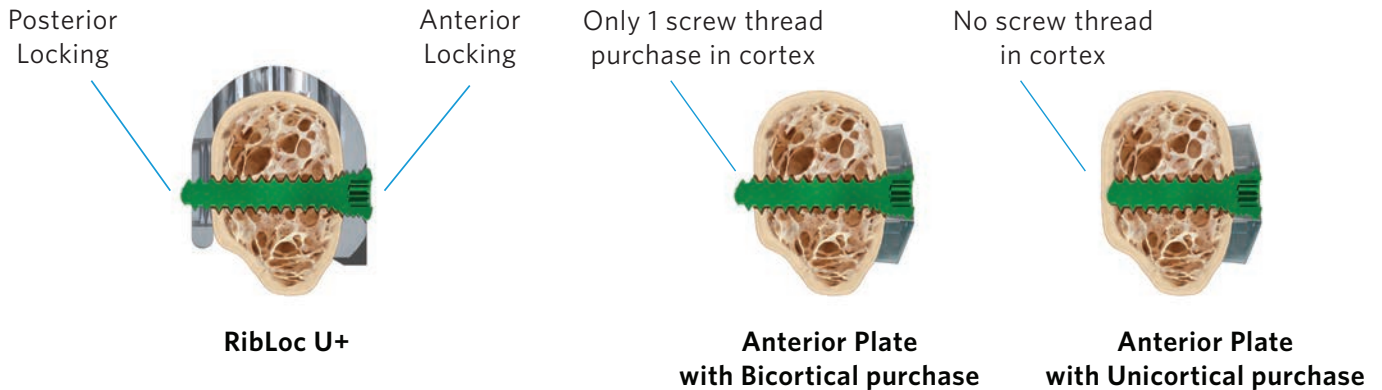


PLATE VARIETY

The RibLoc U Plus system offers U-shaped plates for rib fixation as well as a straight plate to stabilize sternum fractures.

The U-shaped plates are available in 50mm, 75mm, 115mm, 155mm, and 215mm. The straight plate has a length of 126mm.

All plates have been designed to work with existing RibLoc U Plus and U Plus 90 instrumentation.

STERNUM FRACTURE FIXATION

The straight plate is a new implant that offers fixation for transverse sternal fractures while using U Plus instrumentation.

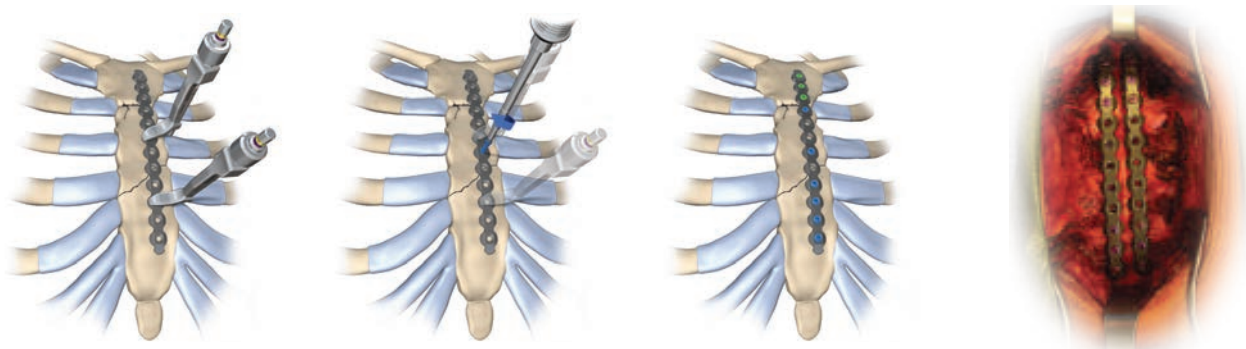


Image courtesy of Bruce Harn, MD, Associate Professor of Surgery, Division of Trauma, Critical Care and Acute Care Surgery, School of Medicine

STRAIGHT-FORWARD INSTALLATION

With the use of a hex driver and drill bit, a rib plate installation can occur in as little as three steps for a simple fracture using the U Plus plate, Primary Guide, and screws.




Compress



Drill



Screw

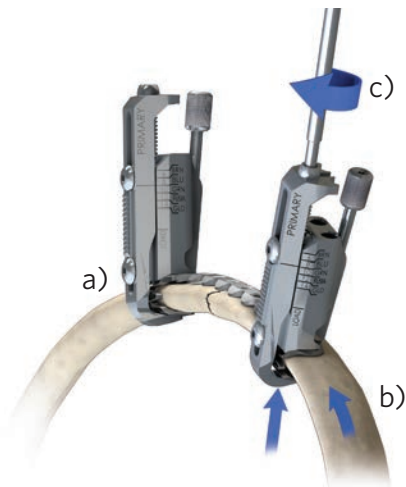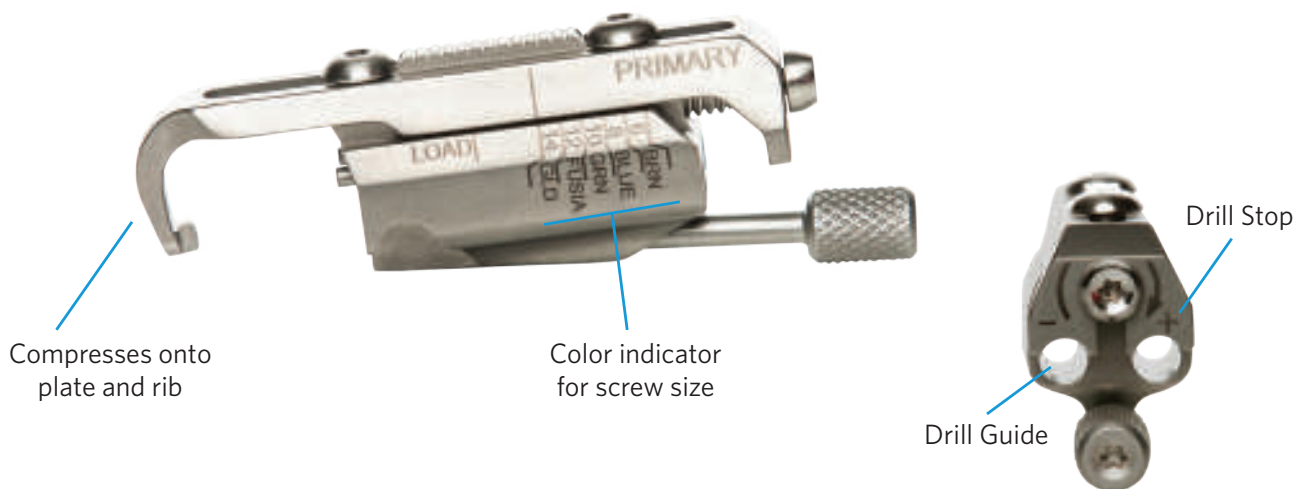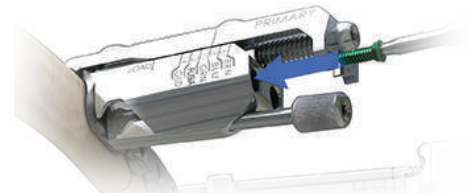

PRIMARY GUIDES ASSIST WITH REDUCTION

- Compress one Primary Guide to hold onto rib.
- To reduce the fracture, grasp the rib and guide it towards the uncompressed Primary Guide (forceps can be helpful).
- Compress the second Primary Guide to hold fracture reduction in place.

Note:

- This technique can also be done using the U Plus 90 Instrumentation.

POWER FOR EFFICIENCY

The 90 degree system was designed for low-profile access while allowing the use of power to compress, drill, and install screws — all with efficiency.

Compress



Drill




Screw



SUB SCAPULA ACCESS

Images courtesy of Jose J Diaz, MD, Professor of Surgery at Shock Trauma Center in Baltimore, MA, University of Maryland.

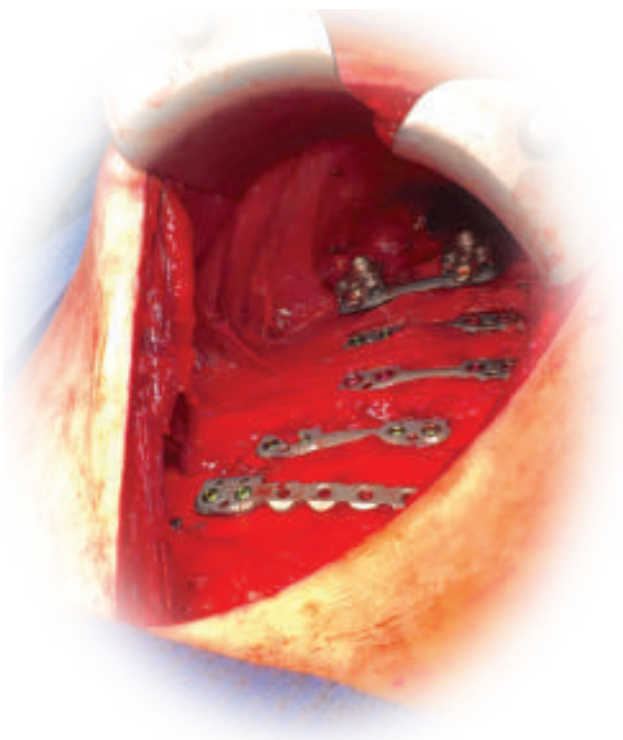

Additional Instrumentation

RibLoc® U+

BENDING TOOLS:



In-plane hand bender



Out-of-plane hand bender



Joystick bender

INTERMEDIATE INSTRUMENTATION:



Intermediate Gauge



6mm 8mm 10mm 12mm 14mm
Intermediate Drill Guides

RIB FORCEPS, DRILL, AND DRIVER:



Rib Forceps



Drill



Driver

RibLoc® U+ 90

DRILL GUIDES, DRILL BITS, AND DRIVER:



Low-profile Drill Guide



Individual Drill Guides and Drill Bits



Driver Bit

RATCHETS AND EXTENSION HANDLE:



Ratchets



Extension Handle

W&H IMPLANTMED, MOTOR AND HANDPIECES:





- 1 Sales JR, Ellis TJ, Gillard J, Liu Q, Chen JC, Ham B, Mayberry JC. Biomechanical testing of a novel, minimally invasive rib fracture plating system. *J Trauma* 2008; 64:1270-1274
- 2 Acute data on file
- 3 Nirula R, Mayberry JC. Rib fracture fixation: controversies and technical challenges. *Am Surg*. 2010 Aug;76(8):793-802
- 4 Sarani B, et al. Pitfalls associated with open reduction and internal fixation of fractured ribs. *Injury* 2015; 46(12):2335-2340

Not all products may currently be available in all markets.

Please also refer to the package insert(s) or other labeling associated with the devices identified in this reference guide for additional information



8055 NE Jacobson Street
Suite 700
Hillsboro, OR 97124

1-866-623-4137

+1-503-686-7200

ACUTE
Innovations® www.acuteinnovations.com

RBL7005_F | Effective 10/2019

an  acumed® company